
# inda®








SEDILE PER DOCCIA  
SHOWER SEAT  
SIÈGE DE DOUCHE  
DUSCHKLAPPSITZ  
DOUCHEZITJE  
ASIENTO MOVIL  
ASSENTO BASCULANTE

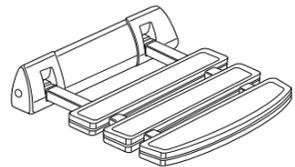## Art. A0436A

1600 N  
~ 160 kg

- I** ISTRUZIONI DI MONTAGGIO
- GB** MOUNTING INSTRUCTIONS
- F** INSTRUCTIONS DE MONTAGE
- D** MONTAGEANWEISUNGEN
- NL** MONTAGEINSTRUCTIES
- E** INSTRUCCIONES DE MONTAJE
- P** INSTRUÇÕES DE MONTAGEM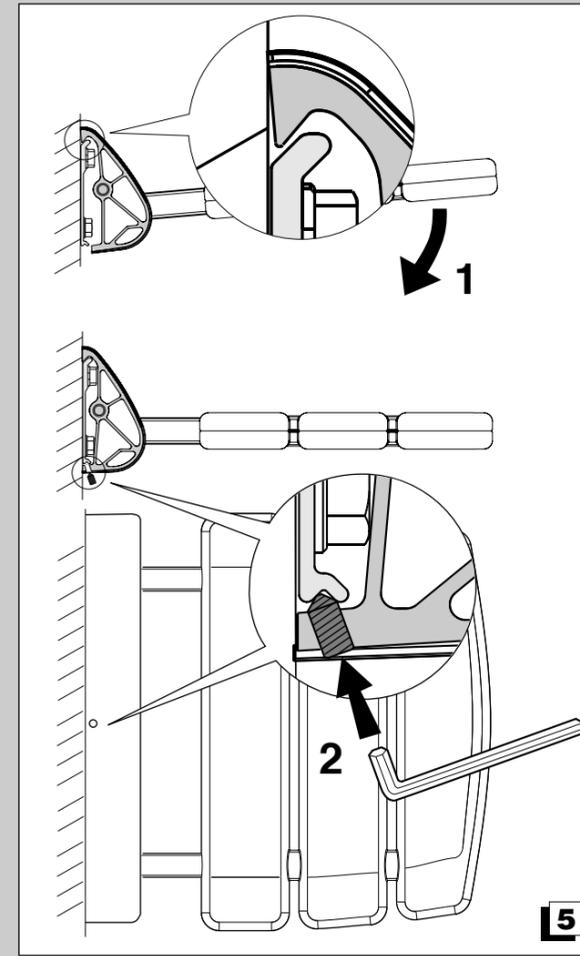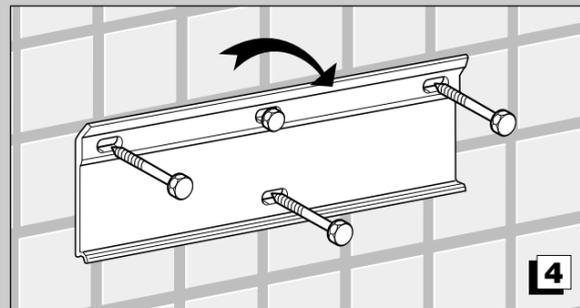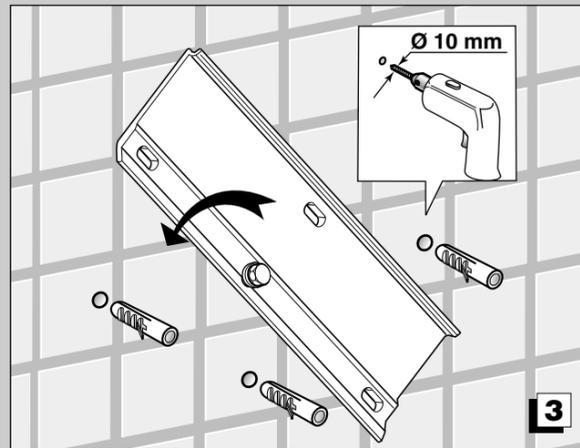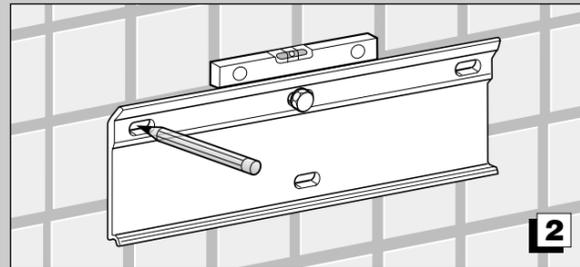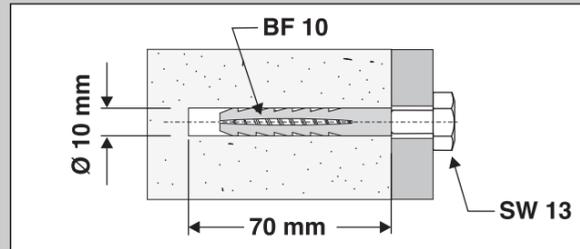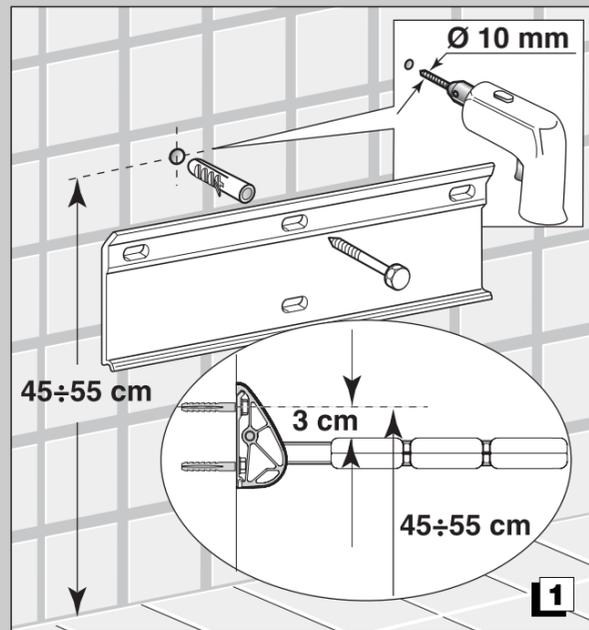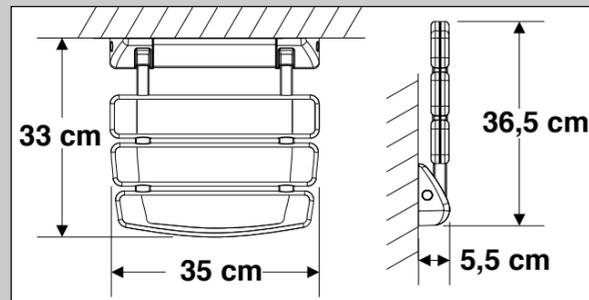

# inda®

INDA srl - Via dell' industria, 41  
24040 Pagazzano (BG)  
Tel. 0039 0363 38061 - Fax 0039 0363 070035  
www.inda.net

11/2015

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**GB****Installation should be performed by specialised personnel****ATTENTION**

- The shower seat has been designed to be fitted inside a shower enclosure or in the bathroom.
- Make sure that the wall onto which the brackets have to be mounted has the necessary features of compactness and resistance.
- The best results can be achieved by installing the shower seat on walls, which are made of concrete or bricks, plastered and eventually lined with tiles.
- Should the wall not be built in concrete or bricks consult with a specialist before the installation.

**FIXING HEIGHT**

- The fixing height of the shower seat depends largely on the user's height. It is usually somewhere between 45 - 55 cm above the bottom of the shower tray or bathroom floor.
- An easy way to determine the fixing height is to measure the seat height of the user's most comfortable chair, and install the shower seat at a similar height.

**INSTALLATION**

- Drill the holes with a bit as shown in the pictures and make sure that they are free from dust before inserting the dowels included in the package (only use these dowels).  
Nylon Plug BF 10  
Wood Screw 8x60 DIN 571- UNI 704  
suitable for Concrete Rc25 9,3 kN

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