
inda®



SEDILO PER VASCA
SEAT FOR BATH TUB
SIÈGE POUR BAGNOIRE
BADEWANNENSITZ
BADZITJE
ASIENTO DE BAÑERA
ASSENTO PARA BANHEIRA

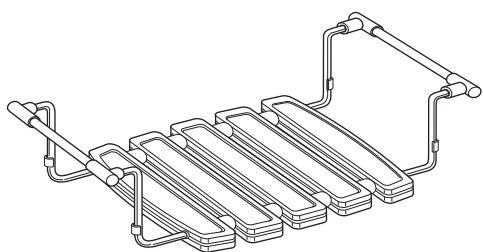Art. A0438A

1300 N
~ 130 kg

- I** ISTRUZIONI DI MONTAGGIO
- GB** MOUNTING INSTRUCTIONS
- F** INSTRUCTIONS DE MONTAGE
- D** MONTAGEANWEISUNGEN
- NL** MONTAGEINSTRUCTIES
- E** INSTRUCCIONES DE MONTAJE
- P** INSTRUÇÕES DE MONTAGEM

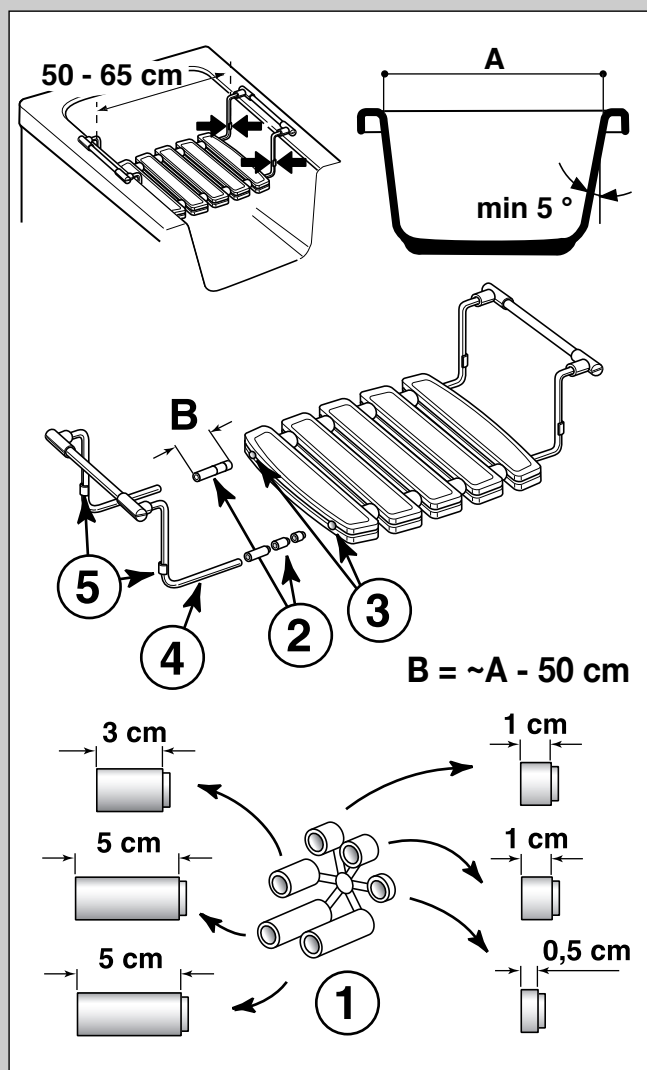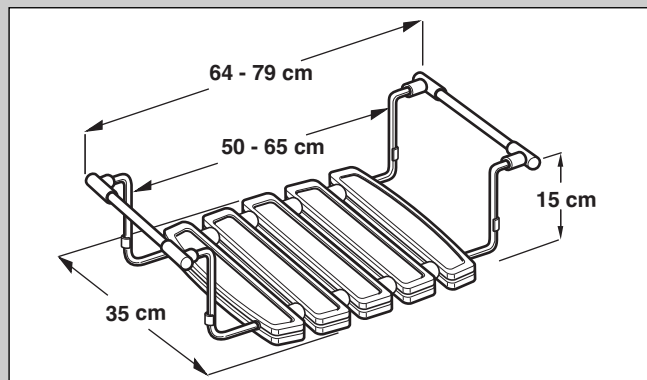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WARRANTY: The validity of the guarantee starts from the purchasing date certified by the seller's stamp and expires after 2 years even if the product has not been used. The guarantee covers all the replacements or repairs due to manufacturing defects (except components subject to wear). The guarantee has no validity if the product has been improperly used.

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GB

ATTENTION

- The bathtub seat is adjustable to suit almost all popular bathtubs. The anti-slip surface made of ABS remains always unchanged as time goes by. The aluminium parts are epoxy-powder painted. The bathtub seat is suitable above all for children and senior people
- **Before use, please check that the bathtub seat is well positionned on all 4 laying points. The bathtubs must be steady (paarticularly the acrylic one)**

INSTALLATION

- Measure the internal width **A** of the bathtub across the upper edge of the same.
- Assemble the distance tubes **1** to a length **B** to be calculated as follows:
B = ~ **A** minus 48 cm
- Insert the assembled tubes groups **2** into the two holes **3** of the bathtub seat.
- **Important: the distance tubes must always be inserted, in order to guarantee the seat stability into the bathtub.**
- Insert the shaped tubular parts **4**.
- Place the seat onto the bathtub and adjust the clips **5** so that the tubular parts correctly lay on the edge.

CLEANING

Quality, technological processes and materials are the features that determine the high standards of our accessories. However, to avoid surface alterations, it is necessary to follow a few guidelines. Clean all finishes exclusively with soap and water, and dry with a soft cloth. Do not use coarse cloths, abrasive products, acids, ammonia products, caustic substances, bleach or solvents. In case of contact with these substances, rinse surfaces immediately and wipe dry. Remember that even fumes can damage metal. Wash accessories next to the toilet bowl at regular intervals to avoid corrosion.

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