

# inda®









SEDILE PER DOCCIA  
SHOWER SEAT  
SIÈGE DE DOUCHE  
DUSCHKLAPPSITZ  
DOUCHEZITJE  
ASIENTO MOVIL  
ASSENTO BASCULANTE

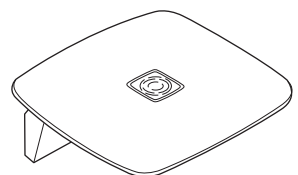## Art. AV036B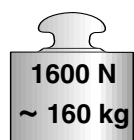

- I** ISTRUZIONI DI MONTAGGIO
- GB** MOUNTING INSTRUCTIONS
- F** INSTRUCTIONS DE MONTAGE
- D** MONTAGEANWEISUNGEN
- NL** MONTAGEINSTRUCTIES
- E** INSTRUCCIONES DE MONTAJE
- P** INSTRUÇÕES DE MONTAGEM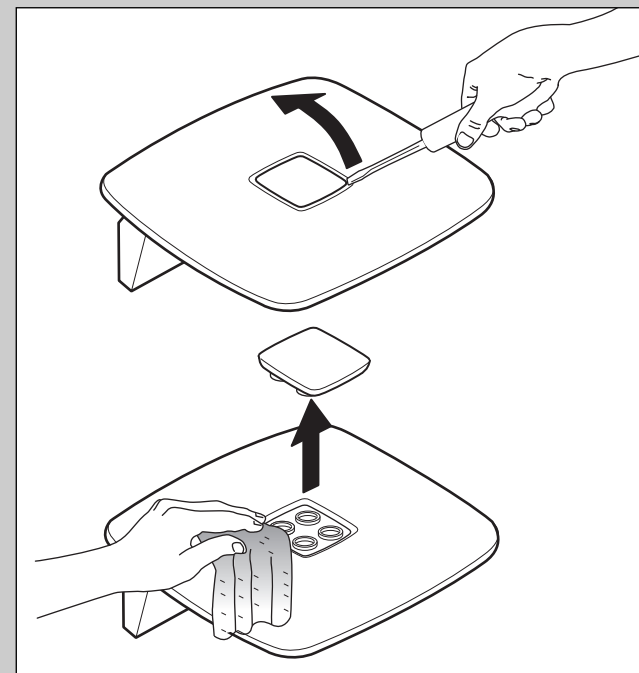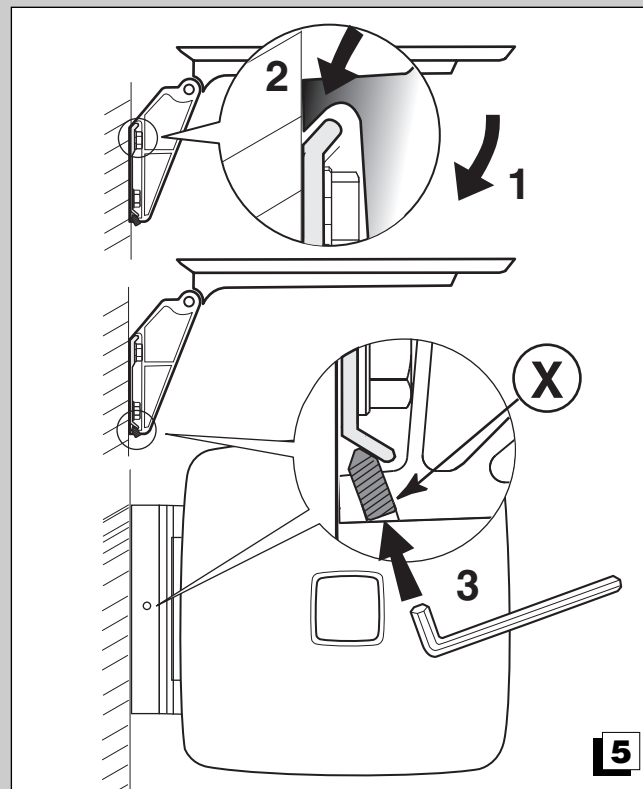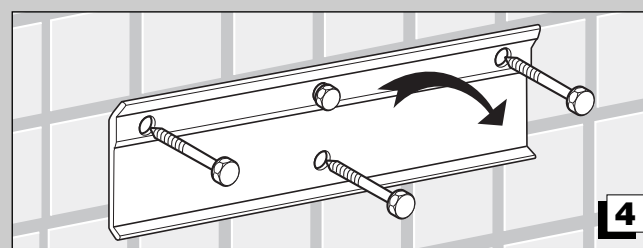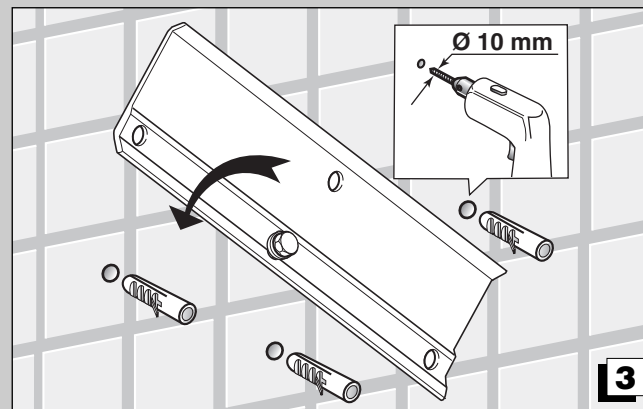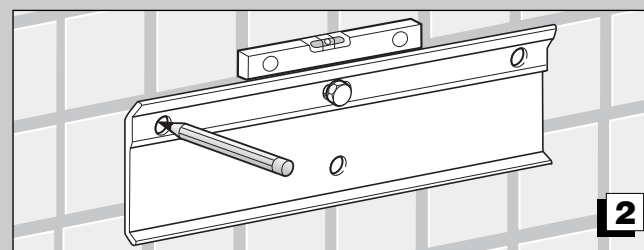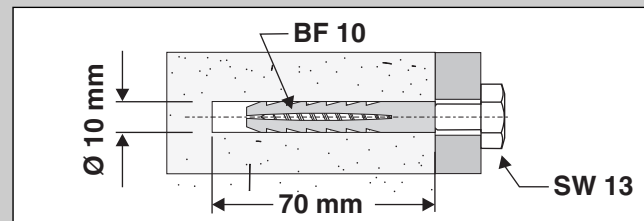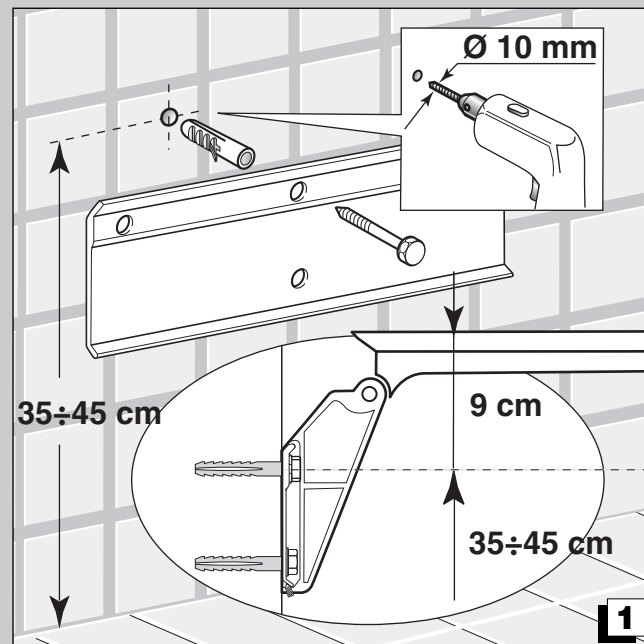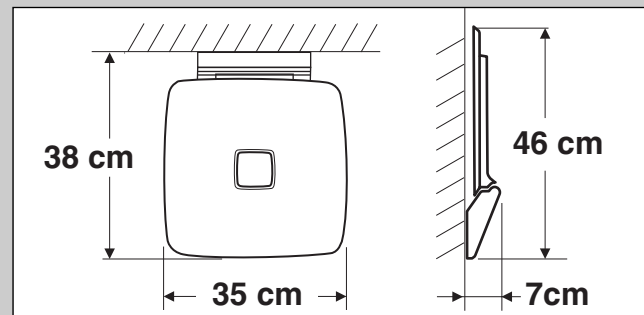

inda®

INDA srl - Via dell' industria, 41  
24040 Pagazzano (BG)  
Tel. 0039 0363 38061  
Fax 0039 0363 070035  
www.inda.net

860V036B

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**GB****Installation should be performed by specialised personnel****ATTENTION**

- The shower seat has been designed to be fitted inside a shower enclosure or in the bathroom.
- Make sure that the wall onto which the brackets have to be mounted has the necessary features of compactness and resistance.
- The best results can be achieved by installing the shower seat on walls, which are made of concrete or bricks, plastered and eventually lined with tiles.
- Should the wall not be built in concrete or bricks consult with a specialist before the installation.
- Frequently check the correct tightening of SETSCREW fixing (X)

- Smooth or damp surfaces can have an adverse effect on the slip resistance
- Do not step onto the seat
- Do not lift with persons on it

**FIXING HEIGHT**

- The fixing height of the shower seat depends largely on the user's height. It is usually somewhere between 40 - 50 cm above the bottom of the shower tray or bathroom floor.
- An easy way to determine the fixing height is to measure the seat height of the user's most comfortable chair, and install the shower seat at a similar height.

**INSTALLATION**

- Drill the holes with a bit as shown in the pictures and make sure that they are free from dust before inserting the dowels included in the package (only use these dowels).  
Nylon Plug BF 10  
Wood Screw 8x60 DIN 571- UNI 704  
suitable for Concrete Rc25 9,3 kN

**CLEANING**

Quality, technological processes and materials are the features that determine the high standards of our accessories.  
However, to avoid surface alterations, it is necessary to follow a few guidelines.

Clean all finishes exclusively with soap and water, and dry with a soft cloth.

Do not use coarse cloths, abrasive products, acids, ammonia products, caustic substances, bleach or solvents.

In case of contact with these substances, rinse surfaces immediately and wipe dry.

Remember that even fumes can damage metal.

Wash accessories next to the toilet bowl at regular intervals to avoid corrosion.

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